

Is your footrest  
long enough?

Extend your footrest with the  
Golden foot extender. Note:  
The extender must be  
ordered with the Chair from  
the factory.



## CAUTION



Do not allow children to play on this mechanized  
furniture or operate the mechanism.  
Leg rest folds down on closing so that a child could  
possibly be injured.  
Always leave in an upright and closed position. Keep  
hands and feet clear of mechanism. Only the occupant  
should operate it.

# Power Lift & Recline Chair

## OWNER'S MANUAL AND LIMITED LIFETIME WARRANTY



The Best Power Lift and Recline Chair Made in the USA 

 **GOLDEN**  
technologies  
YOUR LIFE IN MOTION

 **GOLDEN**  
technologies  
YOUR LIFE IN MOTION

CH-OWN-080105

## IMPORTANT!

READ THIS MANUAL FIRST BEFORE INSTALLING OR OPERATING YOUR POWER LIFT AND RECLINE CHAIR.

Your power lift and recline chair has been engineered from the leg levelers up to provide the ultimate in strength and reliability.

Great care has been taken to provide you with total comfort and safety. Golden Technologies thoroughly inspects and tests each unit prior to shipment.

Your Golden power lift chair is a piece of medical equipment. Therefore, you are required to exercise caution when operating it to ensure your personal safety and that of others around you. The following are rules for the safe operation of your Golden power lift chair.

### ⚠ WARNING

- ⚠ Care should be taken in the unpacking of this unit, so as not to cause injury to persons or damage to the unit. Dispose of all plastic bags and covers, as they can be a smothering hazard to small children.
- ⚠ Do not allow children or pets near the chair while chair is operating.
- ⚠ Do not allow children to play with or operate the chair at any time.
- ⚠ Hand control should be unplugged and placed out of reach of children when chair is not in use.
- ⚠ Do not flop down into the chair when getting into it, or while it is in its lift position.
- ⚠ Do not allow anyone to sit on your lap, or on the arm of the chair, or on the footrest while operating the chair.
- ⚠ Do not place fingers or hands between seat cushion, and the inside arm of chair while sitting in or the chair is in operation or plugged into a power source.
- ⚠ Do not retrieve fallen objects from inside or under the chair while chair is plugged into a power source.
- ⚠ Do not operate chair if there are any obstructions under or in front of the footrest, under the chair or behind the chair.
- ⚠ Do not smoke while operating, sitting or resting in the chair.
- ⚠ To reduce risk of electrical shock, do not immerse any of the electrical components into a liquid, or spill any liquids onto electrical components. Do not open any of the electrical components. There are no serviceable parts.
- ⚠ Warranty will be void if any of the electrical components are opened by other than a qualified service person.
- ⚠ Make sure the chair is placed at the specified distance from a wall/corner (page 4). Failure to do so can damage the chair and will void the warranty.
- ⚠ Do not use any type of electrical extension cord to plug your chair into an electrical outlet.

**GOLDEN**  
Technologies

GOLDEN TECHNOLOGIES  
401 BRIDGE ST  
OLD FORGE PA 18518-2323

PLACE  
STAMP  
HERE

## CONSUMER LIMITED LIFETIME AND SEVEN YEAR WARRANTY

**GOLDEN**, through its authorized distributor / service organization, warrants this chair to original consumer only, against defects in material or workmanship as follows:

### LIFETIME WARRANTY

**GOLDEN**, warrants for the lifetime of the chair by the original consumer to repair or replace at their option the Lift Mechanism, Chair Frame and Recline Mechanism. Labor to replace these parts is covered on all models for three years. The Capri, Slimline and Monarch are limited to six months, The Space Saver is limited to one year, parts and labor on the lift frame, recline mechanism and wood frame.

**GOLDEN**, shall not be responsible or obligated to pay for any cost or expense for examination of any warranted parts nor is Golden to pay for costs for transportation, handling and packaging of the parts or components.

### THREE YEAR PARTS WARRANTY

**GOLDEN**, warrants the electrical parts of this chair against defects to original consumer only, for a period of three years from the date of original purchase by a consumer. The Monarch chair has electrical parts warranty of 2 years. The Capri and Slimline chairs are limited to 1 year electrical parts warranty. Labor to replace these parts is covered for one year on all models except Capri and Monarch, which are limited to 6 months.

**GOLDEN**, shall not be responsible or obligated to pay for any costs or expenses for examination of any Warranted parts nor is Golden to pay cost for transportation, handling and packaging of the parts or components.

### LIMITED WARRANTY FORTH THROUGH SEVENTH YEAR

During the period beginning with the expiration of the full three year parts warranty offered above and continuing through the seventh year after date of original purchase by a consumer, **GOLDEN** will warrant the electrical parts on the following pro-rated basis. Year 4 earns a 35% discount off suggested retail price. Year 5 earns a 25% discount, year 6 a 15% discount and year 7 a 5% discount. By way of example, a part costing \$100.00 that needs replacement in year 4 would cost \$65.00. **GOLDEN** shall not be responsible or obligated to pay for any costs or expense for examination and / or service of any warranted part or to pay for costs of transportation, handling and packaging of the chair and / or parts or components. Capri, Slimline, Monarch and Spacer Saver are excluded from this portion of the warranty.

### TERMS APPLICABLE TO WARRANTY

**GOLDEN** will, at its option, repair and / or replace the defective parts and / or components provided that the warranty parts have not been subject to misuse, abuse or improper service by the consumer, have not been damaged in transit or handling and / or have not been altered or repaired by unauthorized representatives. This warranty does not include costs for unnecessary service calls, i. e., in-home service calls solely for the purpose of educating the consumer about the product, or finding an unsatisfactory power connection, etc. Please refer to weight capacity chart for specifics.

Jr. Petite & Small Models .....	300 lbs.	Capri & Slim Line .....	325 lbs.
Medium, Large & Tall Models .....	375 lbs.	Space Saver Model ..	250 lbs.
Dual Motor 26" & 28" Wide Models .	500 lbs.	New Yorker .....	375 lbs.
Pioneer .....	375 lbs.	Ambassador .....	375 lbs.

**In the event that the product does not conform to this warranty at any time while this warranty is in effect, Golden will correct the defect in conformity with the Consumer Limited Lifetime and Seven Year Warranty. THE LIMITED LIFETIME AND SEVEN YEAR WARRANTY SET FORTH ABOVE IS THE SOLE AND ENTIRE WARRANTY PERTAINING TO THE PRODUCT AND IN LIEU OF AND EXCLUDES ALL OTHER WARRANTIES OF ANY NATURE WHATSOEVER, WHETHER EXPRESSED, IMPLIED, OR ARISING BY OPERATION OF LAW INCLUDING, BUT NOT LIMITED TO ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY DOES NOT COVER OR PROVIDE FOR THE REIMBURSEMENT OR PAYMENT OF INCIDENTAL OR CONSEQUENTIAL DAMAGES.**

**Some states do not allow this exclusion or limitation of incidental damages so the above limitation or Exclusion may not apply to you.**

**GOLDEN** shall not be liable for any incidental or consequential damage or loss not contained in this Warranty.


This warranty gives you specific legal rights, and you may also have other rights, which may vary from State to state.

The in-home service portion of this warranty is valid in all 50 states, the Canadian Provinces, and Puerto Rico.



## Installation Of Your Golden Power Lift Chair

Models: PR-501M26, PR-501L26, PR-501T28

1. Install two 9 volt alkaline batteries in the inline battery holder on the power cable as shown.

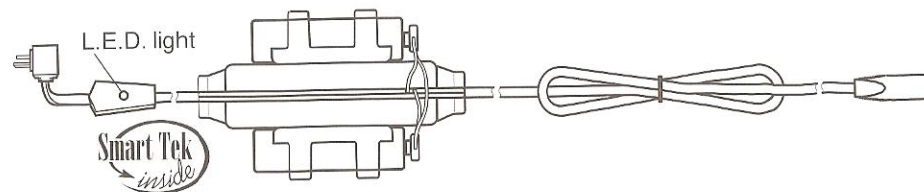

2. Remove the transformer from the transformer box, install two 9 volt batteries and plug the chair into a 115-120V 60Hz A.C. as shown in steps 1 and 2 on page 4.
3. Pull the hand control out of the magazine pocket and secure the cord into the quick release cable clamp located above the pocket on the arm of the chair.
4. Back installation: Press the down arrow on the hand control until the footrest is about parallel to the floor. This will expose the back brackets at the rear of chair.
5. Slide the back down between the brackets until the threaded studs on the back slip thru the holes in the brackets as shown.
6. Thread the flange lock nuts onto the studs and tighten securely using the supplied wrench or a 7/16" box wrench. Return the chair to the seated upright position by pressing the up arrow on the hand control.
7. Attach the bottom of the outside back to the Velcro on the arm cross rail.

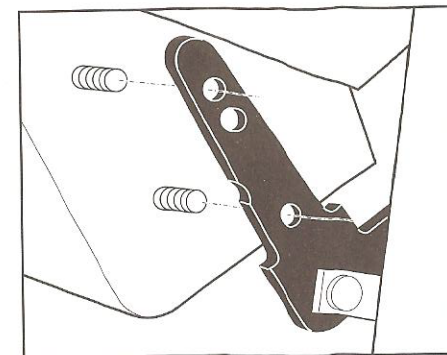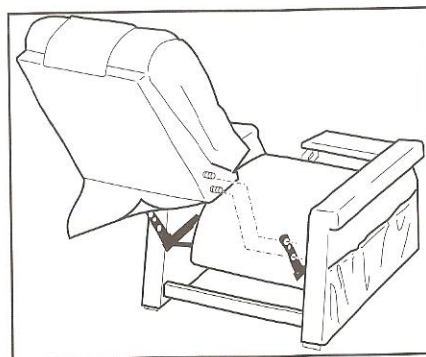

8. Place the chair in the desired position in the room, leaving the chair a safe distance from the wall. See distance chart on page 4 for the proper distance from the wall and corners.
9. Adjust the leg levelers on the bottom of the chair if necessary. Some high pile carpets can obstruct the footrest operation by rubbing against it. Raise the levelers to a height above the pile.



Dear Valued Customer,

Thank you for purchasing this Golden Technologies Power Lift and Recline Chair. ETL and CSA certifications insure that our chairs are the safest available. We believe that this product is made

from the highest quality components available, is engineered to give you maximum service with minimum maintenance, and is backed up with the strongest warranty in the industry. We hope that you enjoy years of pleasurable use of your Golden Technologies chair and that this chair becomes a true assist to you in both sitting down and standing up.

Please take a few moments to review this manual to become familiar with the operation of this chair and our warranty. If you need additional information, please call the dealer from whom you purchased this chair.

Also, please note the following safety information: This power lift and recline chair is a piece of medical equipment designed to help you sit down and stand up. Children should not be allowed to operate it without adult supervision. When not in use, the chair should be kept in the sitting position.

Again, thank you for selecting this Golden chair.

Sincerely,

Robert Golden  
President



## Inquire about these fine mobility products also available from Golden Technologies



**Try our compact luxury scooter!** Our Companion features our patented adjustable tiller, complete self-diagnostics, bright head/tail lights and a very comfortable seat with a weight capacity of 300 lbs!



**Looking for a full time portable scooter?** Our Buzzaround is our smallest scooter offering the same rugged 300 lb weight capacity as the Companion I yet it's lighter! Available in 3 Colors: Red/Blue/Gold



**Our Alanté is the best entry level power wheelchair.** This power wheelchair offers the most value with an oversized captain's seat, retractable desk arm and much more! If you're looking to make your life easier, try our Golden Alanté power wheelchair! Available in 4 Colors: Red/Black/Blue/Sahara




**Are you having trouble sleeping?** Sleep better in the Golden luxury adjustable bed featuring multiple head and foot positioning with the option of a massage upgrade! No more tossing and turning in the Golden Luxury Adjustable Bed! **Not intended to be used as a medical device.**




## Installation Of Your Golden Power Lift Chair

Models: PR-Capri, Slimline, PR-355M, PR-355L, PR-501JP, PR-501S, PR-501S-XW, PR-501M, PR-501L, PR-501T, PR-640, PR-641, PR-747, PR-751TY, PR-752, PR-759, PR-760, PR-906

1. Remove the transformer from the transformer box and install two 9 volt alkaline batteries into the sockets on the transformer.
2. Plug the transformer into a 115-120V 60Hz A.C. outlet and connect the power cable from the chair into the transformer as shown.

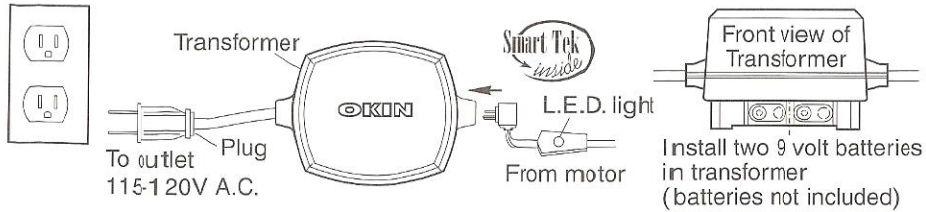

3. Place the chair in the desired position in the room, keeping it 17" away from the wall, if placing the chair into a corner keep it 17" from the wall as shown. PR-501M shown, see distance chart below for your model.

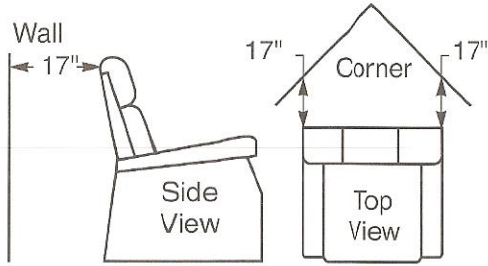

### Distance Required From Wall When Reclined

Model Number	Wall	Corner	Model Number	Wall	Corner
PR-Capri/Slimline	11 in.	11 in.	PR-501L-26D	18 in.	18 in.
PR-355M	17 in.	17 in.	PR-501T-28D	19 in.	19 in.
PR-355L	18 in.	18 in.	PR-640	18 in.	18 in.
PR-501JP	17 in.	17 in.	PR-641	18 in.	18 in.
PR-501S	17 in.	17 in.	PR-747	18 in.	18 in.
PR-501S-XW	17 in.	17 in.	PR-751	18 in.	18 in.
PR-501M	17 in.	17 in.	PR-752	19 in.	19 in.
PR-501L	18 in.	18 in.	PR-759	18 in.	18 in.
PR-501T	19 in.	19 in.	PR-760	19 in.	19 in.
PR-501M-26D	17 in.	17 in.	PR-906	6 in.	6 in.

4. Adjust the leg levelers on the bottom of the chair if necessary to level the chair. Some high pile carpets can obstruct the footrest operation by rubbing against it. Raise the levelers to a height above the pile.
5. Slide the hand control from between the back and arm out toward the back of the chair, secure the cord into the quick release cable clamp located above the magazine pocket on the arm of the chair.

## GOLDEN LIFETIME LIMITED WARRANTY

Register your GOLDEN POWER LIFT CHAIR by filling out this warranty card within 30 days of purchase date and returning to GOLDEN TECHNOLOGIES. Be sure to keep copies of this registration so you have all the information to refer back in case you experience a problem with your chair.

Name \_\_\_\_\_

Address \_\_\_\_\_

City \_\_\_\_\_ State \_\_\_\_\_ Zip \_\_\_\_\_

Purchase Date \_\_\_\_\_ Model No. & Size \_\_\_\_\_

Age of User \_\_\_\_\_ Who Purchased Chair? \_\_\_\_\_

Dealer's Name and Address \_\_\_\_\_

Why Did You Choose The Golden Power Lift Chair?

Appearance      Price      Dealer      Reputation

Additional Comments \_\_\_\_\_


Remember to read your owner's manual carefully before you operate your chair. If you need help or additional information please contact your dealer.

SERIAL NUMBER \_\_\_\_\_



## Power Lift Chair Operation All Models

1. **To recline your chair**, press the down button ( S2 ) and hold until you reach your desired position. Then release the button.
2. **To return to a seated position from recline**, press the up button ( S1 ) and hold until the chair returns to a closed seated position. Then release the button.
3. **To lift your chair**, press the up button ( S1 ) and hold until you reach your desired height. Then release the button.
4. **To return to a seated position from lift**, press the down button ( S2 ) and hold until the chair returns to the floor seated position. Then release the button.
5. **To lift your chair from a reclined position**, press the up button ( S1 ) and hold, the chair will keep going past the seated position and start to lift, release the button when the chair reaches the desired height.
6. **To recline your chair from the lift position**, press the down button ( S2 ) and hold, the chair will keep going past the seated position and start to recline, release the button when the chair reaches the desired reclined position.
7. **Getting into your lift chair.** While standing next to your chair, press the chair up button ( S1 ) and raise the chair to a height you can sit against. This height will vary from person to person depending on the individual's needs. Raise the chair high enough so you do not have to flop down into the chair. **Flopping down into the chair can cause personal injury and damage to the chair.** Press the chair down button ( S2 ). As the chair continues downward, adjust yourself into a position that is comfortable to you. Continue pressing the button until chair is in the seated position ( chair firmly on the floor ), then release the button.
8. **Getting out of your chair**, press the chair up button ( S1 ) and raise the chair to a height that enables you to easily stand. Release the button. Return the chair to the seated position ( chair firmly on the floor ) after getting out of the chair so children, pets or obstructions do not get under the chair.

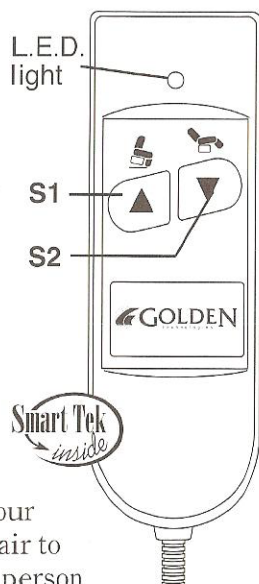

## Battery Back Up System

Your Golden power lift chair comes with a battery back up system so the chair can return to the seated position in the event of a household power failure. The batteries have enough energy to bring the chair to a seated position from either recline or lift. The batteries do not have enough energy nor are they intended to operate the chair for a prolonged period of time in the absence of standard household voltage (120 volt, 60 Hz). Batteries should be replaced after one use. Batteries should be changed yearly.

## Smart Tek (Power Indication)

Regular servicing is not necessary on your Golden power lift chair. Your power lift chair is equipped with an indicator light on the hand control. This light illuminates green when any button on the hand control is pressed indicating power to the hand control.

Your power lift chair also comes with a power indicator light on the power cord connecting the motor to the transformer. The light illuminates green constantly indicating power from the outlet to the motor. The lights have been provided to help you identify electrical disconnects, should they occur, and to help your dealer to diagnose electrical problems, should they occur.

Should the chair not operate and the lights are out, check all electrical connections to the chair. Make sure everything is plugged in properly and there is power to the outlet into which the chair is plugged (see installation on page 4 and 5 for electrical connections). Should you ever encounter a problem with the operation of your power lift chair and the lights are on, such as mechanical problems or one light is out, discontinue all use of the product upon discovery of the problem. Contact the dealer from whom you purchased your chair. They will be able to resolve problems and take care of proper factory service if needed. You will need to be prepared to answer some basic questions about your power lift chair, such as model and serial number. The model and serial number are located on the back of the hand control and the rear of the lift frame. Your dealer will ask you questions about your power indicator lights, such as which one is out. Prior to calling your dealer check all the electrical connections to the chair. Make sure everything is plugged in properly and there is power to the outlet into which the chair is plugged (see installation on page 4 and 5 for electrical connections).

